


6-4-2. Changing Focus Modes

You can switch to the Single Focus Mode if you want to try manually focusing on the image or switch to the Auto Focus Mode if you want CANDY5 HD II to focus automatically.

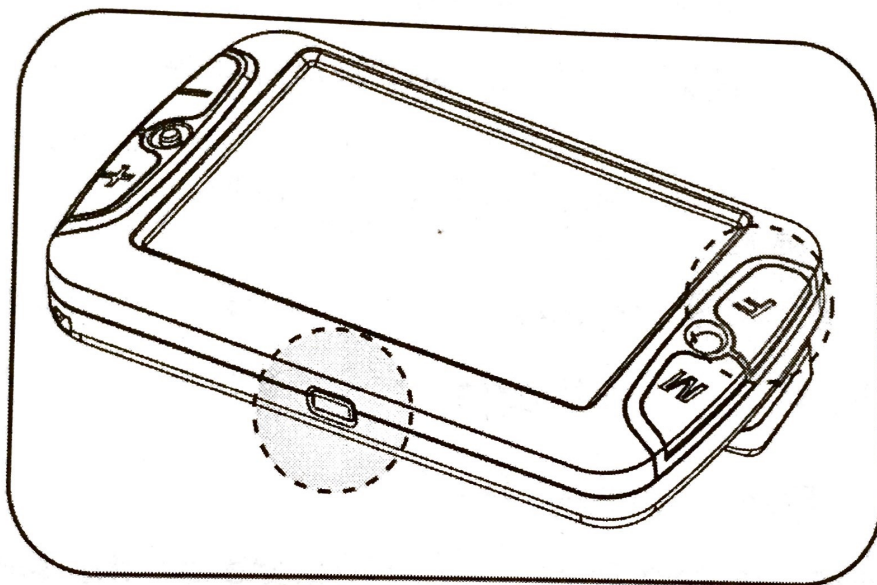


- Single Focus Mode



[Single Focus]

While in Auto Focus Mode, press the **Focus button** while holding the **Freeze button**, to activate the Single Focus Mode. While in Single Focus Mode, the CANDY5 HD II will focus only when you press the **Focus button**.

- **Auto Focus Mode**

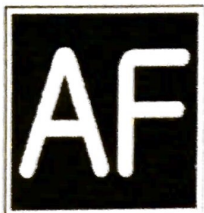

[Auto Focus]


While in Single Focus Mode, press the **Focus** button while holding the **Freeze** button, to activate the Auto Focus Mode. While in Auto Focus Mode, the CANDY5 HD II will focus on the image automatically.

Note: The default setting is Auto Focus Mode.

6-5. Freeze/Save Image

6-5-1. Freeze Image


- There are two **Freeze** buttons: one on the side of the LCD screen and the other on the handle.

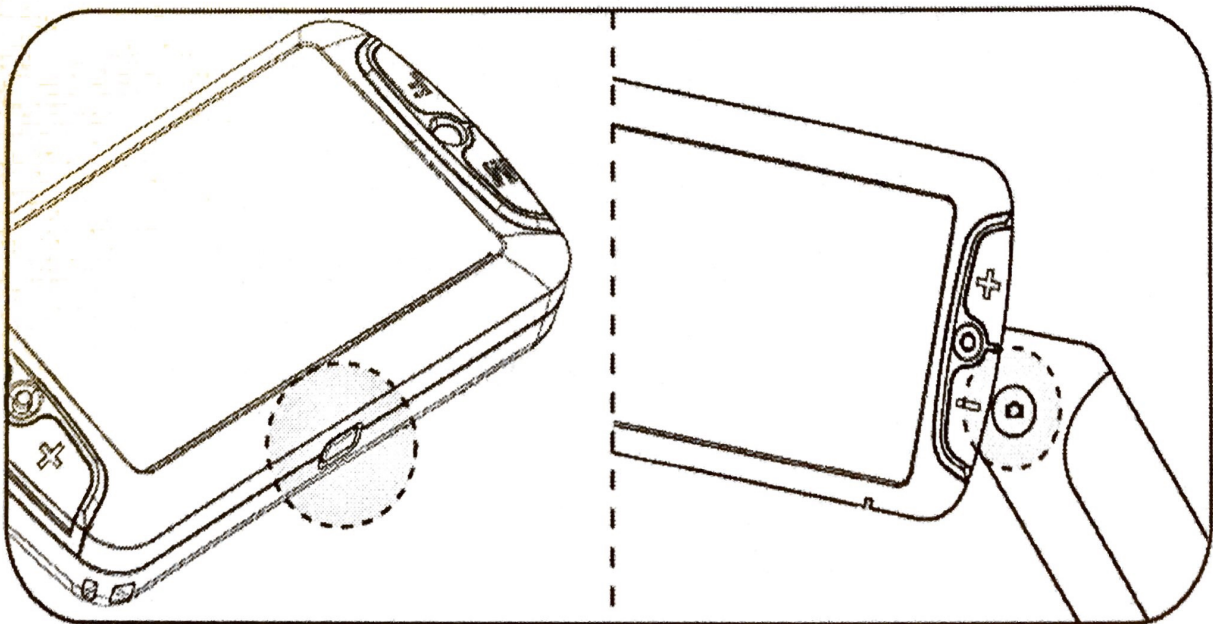

- Press and release the **Freeze button** to take a still shot. CANDY5 HD II freezes the still shot when the button is released.
- Press the **Freeze button** again to release the still shot and return to the live camera mode.

6-5-2. Save Image

- When viewing a frozen image, press the **Freeze button** for more than 2 seconds. Then the frozen image is saved to memory. The icon below is shown when the image is being saved.

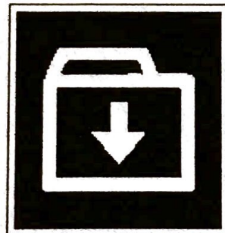


[Saving Images]


- Only **5** images can be saved. If another image is saved, the oldest saved image is automatically deleted.

6-5-3. Viewing Saved Images

- Press the **Color Mode** button and **Focus** button together to view the saved images.

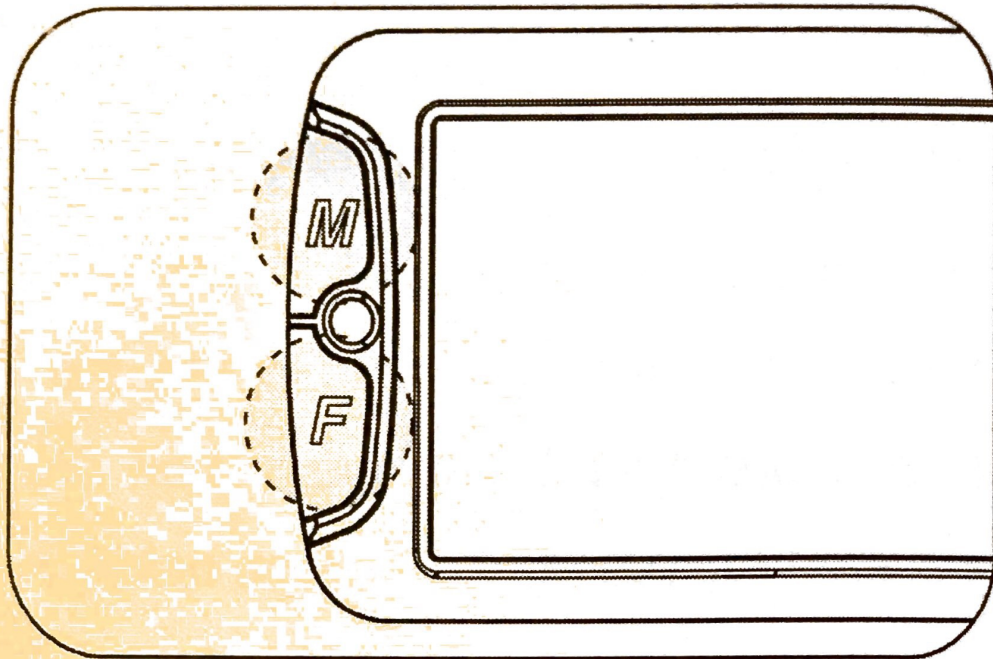

- If you then continually press the **Focus** button, you can cycle through the saved images in the order in which they were saved.
- If there are no saved images, a “No Saved Images” icon is displayed.

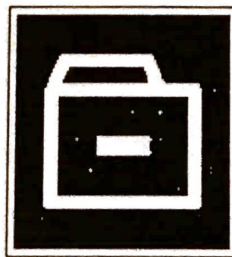

[No Saved Images]

- While viewing a saved image, you can magnify it by pressing the **Zoom In** button.